HYDRA•VACUUM

MANUAL DEL USUARIO
USER MANUAL

HYDRA·VACUUM

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HYDRA·VACUUM

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Dear user,

Thank you for choosing the products manufactured by our company. To facilitate their use and maintenance, we provide you with detailed instructions on how to use them correctly.

- Please read the instructions for use carefully, and save this User Manual for subsequent consultation.
- Follow the instructions carefully to ensure that the equipment is used correctly.
- Do not eliminate or replace any accessory of the equipment.
- Anyone not authorised by the company is prohibited from opening up and handling the components of the equipment.
- Anyone not authorised by the company is prohibited from making changes to or performing maintenance on the equipment. A breach of these rules will effectively cancel the warranty given by the manufacturer or distributors.
- Contact Utsukusy Cosmetics in the event of any breakdown or problem during use.

With HYDRA·VACUUM, we would like to offer professionals a reliable and effective piece of equipment for aesthetic treatments for the face.

UTSUKUSY and its authorised distributors would like to thank you for placing your trust in us by acquiring HYDRA·VACUUM.

1. WARNINGS AND RECOMMENDATIONS

- › UTSUKUSY COSMETICS reserves the right to modify this document, whether partially or completely, without prior notice.
- › All rights reserved. This document may not be reproduced or duplicated, either partially or completely, without the written consent of UTSUKUSY COSMETICS.
- › The treatments and other examples given in this document indicate the characteristics of the described product. UTSUKUSY COSMETICS does not assume any liability regarding any claim related to the properties of those products or regarding any results of the application based on the examples described in the document.
- › HYDRA·VACUUM must only be used by personnel who have been previously authorised and trained by UTSUKUSY COSMETICS or by authorised distributors of the brand.
- › Inspect the packaging to ensure that it has not sustained any major impact and that the equipment does not show any external anomalies due to transport. If there were any major damage, UTSUKUSY COSMETICS or the authorised distributor must be informed within a period of no more than 48 hours. No claims after said period will be admitted.
- › Avoid adverse environmental conditions during use, transport, handling and storage, such as dust, excessive humidity, direct sunlight, vibrations, etc. Do not expose to excessive temperatures, and do not install the equipment near heat sources. Work at a temperature of between 10°C and 30°C and at a relative humidity of between 30% and 75%.

- › Avoid installing the equipment in hydrotherapy rooms or zones where the floor could be wet due to spilled liquids. UTSUKUSY COSMETICS may not be held liable for damages in any of these cases, even during the equipment's warranty period.
- › Do not use HYDRA·VACUUM on patients who are already connected to another piece of equipment.
- › Avoid using HYDRA·VACUUM in the vicinity of another piece of equipment that is generating emissions. It could cause electromagnetic interferences that affect the equipment. Verify the certification referring to the electromagnetic compatibilities of other systems within the proximity of the equipment.
- › UTSUKUSY COSMETICS recommends an annual revision, which should be performed by the technical service of UTSUKUSY COSMETICS or authorised distributors. The warranty of any equipment that is manipulated without the express consent of UTSUKUSY COSMETICS during the warranty period will be invalidated.
- › The incorrect use and upkeep of the treatment applicators and electrodes could cause modifications in the emissions thereof, thereby leading to results that are less than expected, as well as a greater incidence of breakdowns.
- › The equipment works connected directly to the mains. The connection to the mains must be made using a triple outlet with a verified earth connection. Electrical installations in poor condition could have an impact on malfunctioning of the equipment and could cause a breakdown of some its components, thereby invalidating the warranty.
- › Before proceeding to clean HYDRA·VACUUM, be sure that the equipment is disconnected from the mains.
- › Only accessories authorised by UTSUKUSY COSMETICS should be used with the equipment. The use of any other accessory not authorised by the company and/or any handling by unauthorised personnel relieves UTSUKUSY COSMETICS from any liability, as well as from assuming the warranty if the warranty period were in force. It is advisable to use the products that are manufactured for using the HYDRA·VACUUM and that are recommended in this manual.
- › Remember to follow the instructions described in this User Manual for proper use and maintenance of the equipment.

2. WARRANTY

- › UTSUKUSY COSMETICS guarantees the equipment against all manufacturing defects during the warranty period, which is valid as from the invoice and delivery date for a period of 1 year.
- › Treatment applicators and patient electrodes are excluded from the warranty period, given that they are subject to wear due to use of the equipment.
- › Annual revisions are excluded from the warranty.
- › In the event that any equipment is handled, adjusted or repaired or any security seals are broken by third parties without the express, written authorisation of UTSUKUSY COSMETICS, the warranty will be void.
- › Any damage to the equipment caused by falls or due to abrasions to the paint caused by liquids and/or substances is excluded from the warranty.
- › Any equipment pending payment and/or for which payment is late is excluded from the warranty.
- › This warranty will not be applicable if there are any indications of use and maintenance other than that which is described in the manual, as well as any indications of defective electrical installations that do not comply with regulations in force.
- › The warranty includes the replacement, at no charge, of the necessary components that have to be replaced, as well as the cost of labour and the cost of transport for pick-up and delivery. The cases described in the preceding points or warranties agreed with the seller are excepted, which must be noted on official documents, such as estimates, contracts or invoices.

3. PRINCIPLES OF THE VACUUM AND COSMETIC COMBINATION

Vacuum therapy is characterised by being a non-invasive and painless treatment that is not dependent on the skin phototype and that can be used any time of year.

Its action mechanism is based on suction or negative pressure and the subsequent vacuum or rest. This action can be performed continuously, with the suction constant, pulsed or regulated. This method is the most-used in aesthetic applications, given that it avoids the possibility of causing greater skin flaccidity, pain during treatment and risks in tissues with minor vascular problems.

One of its main advantages is that it lacks a thermal effect, given that the suction is mainly mechanical on the tissue. Therefore, its secondary effects, as well as its contraindications, are reduced significantly in comparison with other technologies. Its mechanical effects due to suction and the vacuum cause vasoconstriction and vasodilation of the tissue, thereby improving the tone of the tissue. There is clear improvement of the circulation and microcirculation. This significantly improves the supply of nutrients to cells and improves oxygenation, which reduces the signs of cellular ageing, given that the mechanical action on the cell will also favour the function of the fibroblasts in improving the collagen and elastin due to the general trophic effect in cellular biochemical changes.

VACUUM COMBINED WITH COSMETICS.

How can we improve the function and results in facial treatments?

Based on the benefits that are obtained due to the vacuum-suction action detailed in the preceding point, we can multiply the effects and increase absorption by applying active substances, whether they hydrate, they are nutritive, they are tensors or they have any other aesthetic purpose.

4. INDICATIONS AND CONTRAINDICATIONS

The use of HYDRA-VACUUM is indicated for the following AESTHETIC APPLICATIONS in the combined VACUUM - COSMETIC application.

FACIAL

HYDRA-VACUUM is indicated for the FOREHEAD – FACIAL OVAL – DOUBLE CHIN – NECK – NECKLINE

- Facial skin tightening
- Fine and medium wrinkles
- Improved skin texture
- Acne. Acne scars
- Normalisation of hyperpigmentation
- Improvement of minor vascular treatment. Spider veins, couperose
- Luminosity and improved hydration and moisturising
- Restoration of scars
- Adherences Capsulitis Post-operative recovery on the breast

The use of HYDRA-VACUUM is contraindicated in the following cases:

- Acute inflammatory states
 - Skin insensitivity (assess the degree)
 - Open wounds or a lack of integrity of the skin in the application zone
 - Paediatric patients weighing less than 15 kg
 - Metastasis and neoplasias
 - Bleeding ulcers
 - Arthritic processes or bone pathologies
 - Mucous membranes
 - Recent scars
 - Severe vascular injuries Varicose veins Thrombophlebitis
- IMPORTANT. In the event of any doubts about skin treatment on suspicious spots, warts, etc., consult a medical professional. This equipment and its treatment are not suitable for pathologies or therapeutic treatments.

INSTRUCTIONS FOR TRANSPORTING THE EQUIPMENT

If the equipment has to be transported, the following aspects must be considered:

- Avoid abrupt movements during transport, and ensure that both the console and the accessories are firmly fastened to prevent vibrations and impacts.
- Avoid very hot or humid environments (levels above 80% relative humidity are not recommendable). Keep the equipment in environments that have adequate ventilation.
- Carefully check both the console and the treatment accessories once the equipment has been put back into position. Be sure that the console connectors correspond to the correct applicator. The connectors therefore have a single valid position (never attempt to force the connectors).

TECHNICAL SPECIFICATIONS

MAXIMUM PRESSURE VACUUM 80 KPA

ELECTRICAL SPECIFICATION 220 V / 50 Hz

POWER 80 VA

STANDARD GB 4706.1-2005


CE LOW-VOLTAGE AND ELECTROMAGNETIC FIELD CONTROL YES

5. TECHNICAL DESCRIPTION AND START-UP

HYDRA·VACUUM is composed of the following elements:

- Support for product dispensers
- Recycling dispenser
- 500-ml product dispenser
- Power connection cable
- Vacuum treatment applicator
- 6-piece hexagonal treatment tips
- 2-mm circular treatment tips
- User Manual

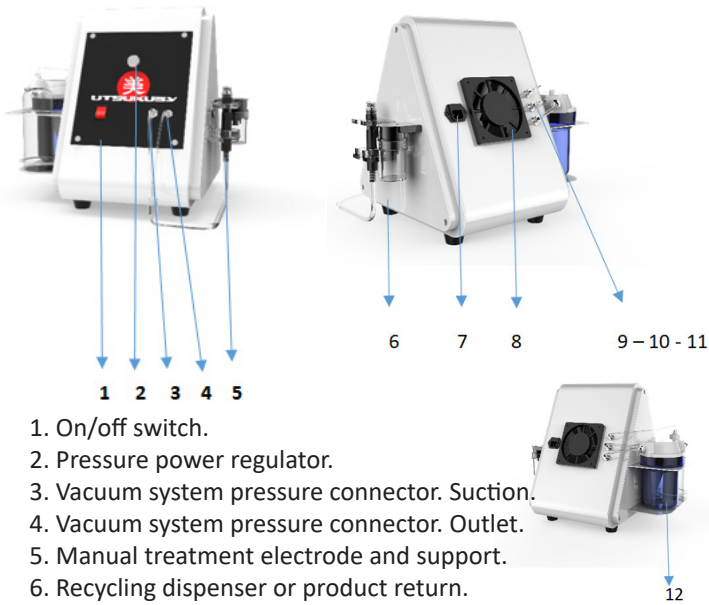Image and description of the equipment's connections

1. On/off switch.
2. Pressure power regulator.
3. Vacuum system pressure connector. Suction.
4. Vacuum system pressure connector. Outlet.
5. Manual treatment electrode and support.
6. Recycling dispenser or product return.
7. Power supply connection.
8. System cooling.
9. Upper vacuum connector of the product dispenser.
10. Middle vacuum connector of the recycling dispenser.
11. Vacuum connector of the treatment applicator.
12. 500-ml active product dispenser.

CONNECTIONS AND OPERATION OF THE SYSTEM

1. Connect the suction pressure tube (No. 3 in the figure) and the pressure outlet tube (No. 4 in the figure) located on the front panel of the equipment. The connection is made by applying slight inward pressure, without forcing the connectors.
2. Also make the connections at the back of the equipment (Nos. 9, 10 and 11 in the figure).
3. Depending on the treatment needs, introduce the desired product and quantity in the dispenser (No. 12 in the figure). Follow the advice of the trainer and the treatment protocols in this manual regarding the amounts to be used and the desired treatment times.
4. Attach the application tip or electrode to the treatment applicator, as required.
5. Connect the power cable at the back of the equipment (No. 7 in the figure).
6. Connect the equipment to the mains.
7. Turn on the on/off switch (No. 1 in the figure).
8. Use the potentiometer located on the front panel of the equipment (No. 2 in the figure), and adjust the desired vacuum pressure.
9. To adjust the product flow, use the regulator located at the intersection of the vacuum tubes. In the horizontal position, the regulator allows maximum product flow, which is recommended for treatments of very short duration, given that product consumption will be greater. In the vertical position, the regulator blocks the flow of product. The regulator should be

adjusted by going from the horizontal position to the vertical, based on the desired flow of product. The selected angle will allow greater or lesser flow.

PRECAUTIONS FOR USE AND MAINTENANCE

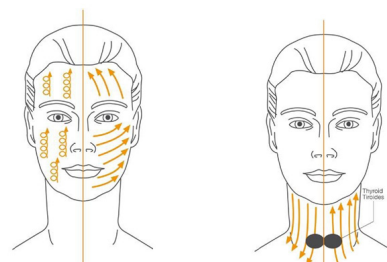
1. When the vacuum is used on the skin, impurities and product sedimentation will accumulate in the desquamation system. The use of liquid products is advised. NEVER use substances with a high molecular weight, such as creams or gels. Using such products could cause the tubes to UTSUKUSY COSMETICS me blocked, as well as cause the internal vacuum pressure pumps to fail.
2. It is advisable to clean the treatment tubes daily using the recommended product, with the suction power at maximum. This suction will take impurities and product sedimentation to the dispenser.
3. Do not use replacement nozzles if they are damaged or show signs of cracking. They could damage the skin.
4. Before using the equipment, check that it is in good condition, including the accessories and the console, and make sure that the power supply cable is intact and shows no signs of damage. Avoid placing the equipment in very warm, humid or dusty environments. Leave a minimum separation of 30 cm between any wall and the console so that the equipment cooling system can work properly.
5. Approximately every 40 hours of use, it is advisable to externally vacuum the rear ventilation grille to prevent possible deposits of dust inside the console and to prevent the equipment from overworking.
6. Do not use the equipment simultaneously with another piece of equipment. There could be incompatibilities.

IMPORTANT:

- Immediately turn off the equipment if any of the following situations occur:
- Liquids are accidentally spilled on the console.
- Abnormal heating of the console, of the power supply cable, of the applicator, etc.
- Unusual or exaggerated sounds from the equipment.
- A strange odour or smoke is noticed.
- Accidental damage to any of the applicators or contact electrodes (due to being dropped, hit, etc.).
- Alterations of the electric power supply (sudden voltage increases).

IMPORTANT: In all of these cases, it is advisable to have the equipment checked by qualified personnel. Contact the manufacturer or an authorised distributor. Do not attempt to repair a breakdown yourself if it has not been previously diagnosed by an official Technical Service. Once the treatment has been completed, it is advisable to turn off the equipment if there is going to be a long period of time until the next treatment. At the end of the workday, it is preferable to disconnect the equipment from the mains by unplugging the power supply cable.

MARKING OF TREATMENT ZONES: FACIAL

INFORMED CONSENT MODEL

AESTHETIC APPLICATIONS

Patient's name:

DNI (National Identity Document). : _____ of full legal age (Yes/No)

- The undersigned declares and confirms that, after having described my wishes and expectations regarding the vacuum and/or cosmetic treatment, I have received sufficient explanations about the treatment and its possible secondary effects.
- I have been informed that the duration of the session will depend on the selected treatment area.
- I have been informed that it is possible that reddening will appear in the treated zone, which will disappear after a few hours without further consequences, so I can perform any usual activity.
- I was explicitly cautioned that I must notify the professional if I have been exposed to UV rays or direct sunlight on the same day as that of treatment or during the preceding days.
- I understand that the results and expectations can vary due to various circumstances, including genetics, environmental circumstances, following the protocol and advice and the condition of the tissue. I will therefore follow the professional's advice.
- I undertake to obey the professional's instructions, regarding both the treatment and the subsequent recommendations between sessions.

To be completed by the client:

- Your state of health is: Excellent / Good / Regular / Deficient
- Have you ever received vacuum treatment and/or vacuum treatment in combination with cosmetic treatment? Yes / No
- Did you suffer any secondary effect after the treatment? Yes / No
- Are you allergic to creams, medications or anything else? Yes / No
- Do you have any skin problems (dermographism, herpes, pruritus, allergies)? Yes / No
- Do you tend to get spots on your skin? Yes / No
- Have you ever had any skin tumour or injury? Yes / No
- Do you have tattoos or micropigmentation? Yes / No
- Are you pregnant? Yes / No
- Have you been diagnosed with epileptic symptoms? Yes / No
- Do you wear a pacemaker or any similar device? Yes / No
- Do you suffer from any cardiac ailment or blood pressure problems? Yes / No
- Do you take any medication regularly? Yes / No

Date:

Signature:

If a minor, the identification, authorisation and approval of the parents or guardians.

HYDRA·VACUUM

+34 93 245 75 76 Fax: +34 93 231 09 22

info@utsukusycosmetics.com

Calle Valencia 629 bajos, CP 08026 Barcelona

www.utsukusycosmetics.com

HYDRA·VACUUM

GARANTÍA: 12 MESES

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**WARRANTY PERIOD : 12 MONTHS WARRANTY
FROM THE DATE OF PURCHASE.**

PROFESSIONAL USE ONLY